

# Blood Temp Indicator 10°C BT10-INDICATOR-TSSO

## Instructions:

- 1 Prior to application of the Blood Temp Indicator, the blood bag must be pre-chilled in a refrigerator (2°C-8°C) to relevant blood bank regulation temperature for 24 hours.
- 2 Indicator should be applied on the lower 1/3 of the blood bag where the greatest amount of blood is held. Area should be clean and free of oil and moisture.
- 3 Prior to arming the Indicator, verify that the arming window is yellow and the breach window is white. The Indicator must be at a temperature above 10°C to arm.
- 4 Firmly squeeze the blister located on top of the Indicator (Figure 1). Verify that the arming window changed color from yellow to green; if not, squeeze the blister again.
- 5 Once the arming window is green (Figure 2), remove the Indicator from the adhesive liner. Adhere the Indicator to the prepared location and apply constant pressure for 3-5 seconds to ensure successful adhesion.
- 6 After the Indicator is adhered to the blood bag, return the blood bag to cold storage until use.
- 7 If the cold chain has been broken, the breach window will turn blue (Figure 3).

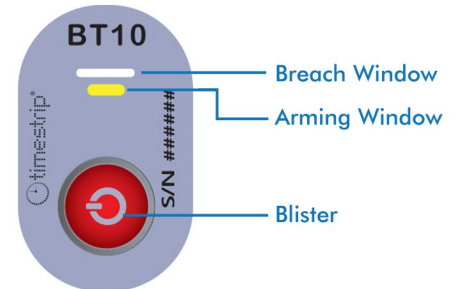


Figure 1

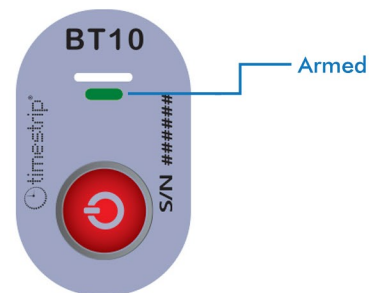


Figure 2

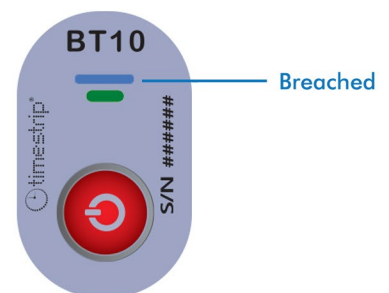


Figure 3

 **DIGI-TRAX**<sup>®</sup>  
identify your future



800-356-6126  
www.digi-trax.com  
info@digi-trax.com  
650 Heathrow Drive  
Lincolnshire, Illinois 60069